Information about the product and the Instructions:

Manufactured by TWM™ –

Tsadik Weiser Medical LTD Otniel 149, Israel.

Patent NO IL 303402.

Approved by the Israel Ministry of Health.

AMR registration number 37440001.

Made In Israel.

Version 1.1

⊗ Don't reuse.

The product must be stored in its original packaging.

- Do not apply the tourniquet on joints.
- Do not apply directly on an injury.
- Do not use the product when it is damaged or broken.
- The product is intended for the professional user,
The instructions for use are not a substitute for instruction.

Limitation of Liability

The company, Tsadik Weiser Medical Ltd, or anyone on its behalf, shall not be held responsible for direct or indirect damage caused because of using the product.

The use of the product is the sole responsibility of the user.

The company bears no obligation and/or responsibility in connection with the use of the product.

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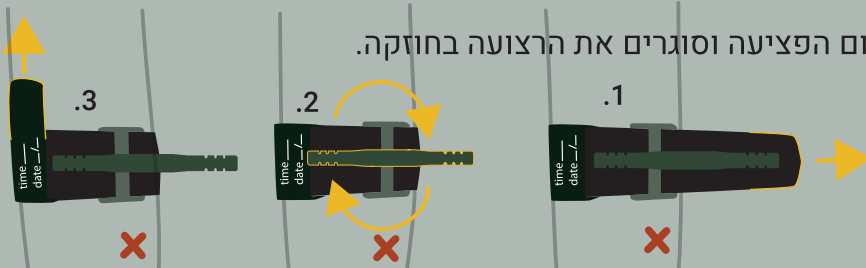
In cases of bleeding from the limb:

1. Place the tourniquet 6 cm (2.5 inches) above the site of injury and close the strip strong.
2. Turn the handle until the bleeding stops.
3. Secure the handle and the strip.

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In cases of bleeding from the shoulder:

1. Connect the pressure pin to the tourniquet, and position it so that it is perpendicular to the neck. Apply manual pressure on the tourniquet until the bleeding stops.
2. Pass the strap through the patient's belt and through the buckle. An additional tourniquet can be used instead of or in addition to the patient's belt.
3. Close the strap tightly. You can turn the handle to increase pressure. Manual pressure should be continued on the tourniquet throughout the evacuation.

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In cases of bleeding from the upper thigh:

1. Connect the pressure pin to the tourniquet and place it over the pelvic bones, with the pressure pin over the femoral artery.
2. Apply manual pressure on the tourniquet until the bleeding stops and close the strap tightly.
3. Turn the handle until the bleeding stops completely and secure it. Continue to exert manual pressure on the tourniquet throughout the evacuation.